

RESCOMF[®]

Heated CPAP Tubing Adapter

User Manual



How to use Heated CPAP Tubing Adapter?

If you have a RESCOMF XD100 Ventilation Disinfecter and a CPAP Machine that uses heated tubing, a heated tubing adapter is needed to connect the XD100 for cleaning your mask and heated tubing, here is a four-step process for your reference:

Step 1: Disconnect the heated tubing from the CPAP machine;



Step 2: Connect the adaptor to the heated tubing, and connect another end of the adaptor to the XD100 (Along the direction of the arrow mark of the adaptor);



Step 3: Place the XD100, tubing and the mask/nasal pillow into an airtight container together.



Step 4: Press the power button of XD100 to begin sterilization of the tubing and the mask. Then cover the lid of the container and it will turn off automatically after sterilization is completed.



The different brands that have heated tubing include:

- ResMed Airsense 10
- ResMed Aircurve 10
- ResMed S9 series
- Philips DreamStation
- Philips Respironics 60 Series
- Fisher & Paykel's ICON Series
- SleepStyle 600 Series

✂ **Heated Tubing Adapter works with all Heated CPAP tubing.**

Note: For better disinfection results it is recommended that you put your CPAP/Bi-Level machine and RESCOMF Ventilation Disinfectant inside an airtight container, the dimension of this container size is smaller or equal to 350x220x140mm(13.78"x8.66"x5.57").

The RESCOMF XD100 is a state of the art CPAP cleaning machine and one of the most respected on the market.